

Silicone Tubing Set,
set up with two 2,800 mL
Perfadex bags.



FOR PERFUSION AND DONOR LUNG PROCUREMENT

- *Rapid flushing during procurement*
- *Simultaneous/continuous perfusion from two Perfadex bags*
- *High flow rates at relatively low pressures*
- *Delivered sterilised and CE-marked*


The Silicone Y Tubing Set is specifically designed for rapid flushing of donor lungs during procurement. The set permits simultaneous/continuous perfusion from two Perfadex PVC-bags, given either together or one at the time, whereby one bag can be exchanged while the other is still flushing.

The wide bore 3/8" tubing permits use of high flow rates at relatively low pressures, to reduce endothelial injury. The use of silicone tubing reduces the risks of kinking.

The tubing set is delivered sterilised and CE-marked.

The hose set for lung perfusion is 200 cm long and consists of two 50 cm tubes which are jointed together by a y-connector to a 150 cm tube. The 50 cm hoses are equipped with spikes and all three tubes are fitted with tubing clamps for regulating the flow. The tubing set is sterile (ethylene oxide) and intended for single use only.

Before a lung transplantation is performed the lungs are irrigated with perfusion fluid. The tubing set for lung perfusion is designed for transferring perfusion fluid from IV bags to the lungs. This shall be carried out via an arterial cannula with a 3/8" connector equipped with a valve.

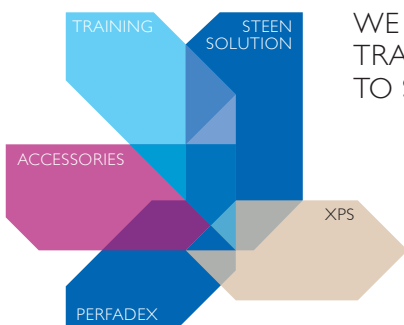

WE EMPOWER
TRANSPLANT TEAMS
TO SAVE MORE LIVES

XVIVO Perfusion AB, Box 53015, SE-400 14 Göteborg, Sweden.
Tel +46 31 78821 50, fax +46 31 78821 69. order@xvivoperfusion.com
www.xvivoperfusion.com

XVIVO
PERFUSION